

Visioffice[®] 2

powered by *Activisu*[®]

June 2014 | Software Update



Visioffice[®] 2 Software Version 2.4.19

This June, the Visioffice 2 system will update to the latest software version, **2.4.19**.

Improvements in this version include a new training mode to help ECPs learn to use the system, recommended speech to use with patients, an improved materials demonstration, navigation enhancements and more!



Flagged Values in Measurements

Now a red circle will display when a measurement value seems incorrect. Tapping the circle will open a dialog box that explains why the system has flagged the value.

The screenshot shows the Visioffice 2 software interface. At the top left is a patient photo. Below it is a table of measurements for the Right Eye (RE) and Left Eye (LE), with a 'Total' column. Red arrows point to the LE 'PD' value (38.8) and the RE 'ERC Distance' value (22.9), both of which have a small red circle with an exclamation mark next to them, indicating they are flagged. Below the table is an 'Add segment height' button. Further down are 'Pantoscopic angle' (11.2°) and 'Head Cape' (5.5°) settings. At the bottom is a 'Frame' section with 'Material' set to 'Plastic' and 'Wrap angle' set to -10.1°. A 'Frame' dialog box is open in the foreground, displaying a warning icon and the text: "Please, check the boxing lines and rim positioning". At the bottom of the dialog are three buttons: 'Redo the measurement', 'Redo the boxing', and 'Keep this measure'.

	RE	LE	Total
PD	22.8	38.8	61.6
Height	20.8	20.6	
ERC Distance	22.9	23.1	

Material: Plastic

Wrap angle: -10.1°



Warning: Please, check the boxing lines and rim positioning

Buttons: Redo the measurement, Redo the boxing, Keep this measure

Training Mode and Recommended Speech for Measuring


To help with **training** opticians to use Visioffice[®] 2, a new **training mode** has been added. When the training mode is turned on through the home screen of the software (indicated by the presence of a question mark in the bubble, image to right), **popup** screens will display **tips** on what needs to be checked as they go through the measurement process. This popup will display for about eight seconds before minimizing. Once the message minimizes, the message can be recalled by touching the arrow at the bottom of the screen.

In addition to this, **suggested speech** to use with patients will now always be seen across the top of the screen during the measuring process.

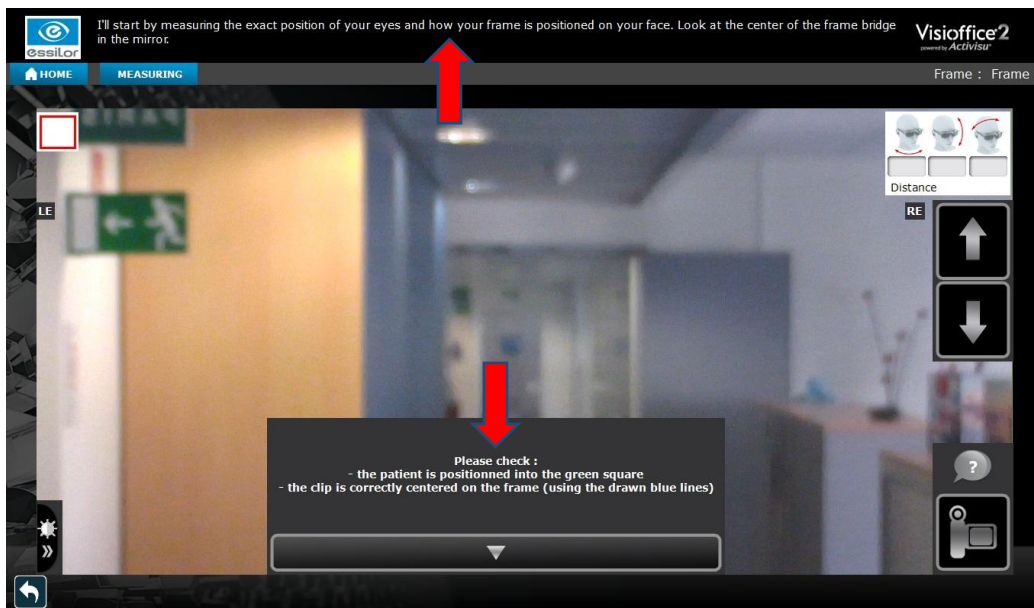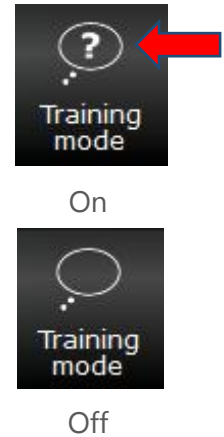

Materials Demo

The materials demo has been upgraded to allow users to compare the lens weight, cosmetic effects of materials on the patient's eyes, and edge and center thickness of materials.

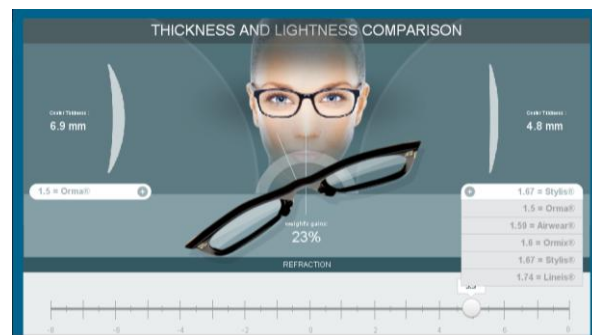


Navigation Improvement



In each part of the system, there is a new button next to the home button that will quickly take the user back to the current area's home screen.


Wheels for Entering Prescriptions

In the patient file screen, **wheels** have been added to enter patient prescriptions more easily.

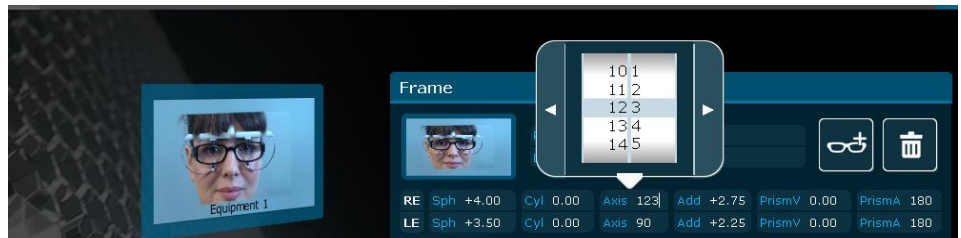

Enable/Disable Buttons

Enable/disable sliders have been added in the measurements page for head cape and fitting height optimization.

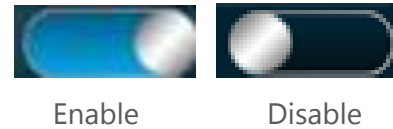

How Do I Get The Update?

It's easy! Once available, the Visioffice 2 system will **automatically begin downloading** the installation package via your active internet connection. Once the software download has completed, a **notification will appear** on the touch screen to guide you through finalizing the software installation. Once this process has completed, you may **open the Visioffice® software** and have access to these new features.

If your Visioffice 2 is not connected to the internet, you will be required to contact the Visioffice Equipment Group to arrange for an offline installation.

Learn More

More information about the Visioffice 2 system and its features can be found online at www.EssilorVisioffice.com.

For any technical issues regarding your Visioffice 2 system, please contact the **Visioffice Equipment Group** at **(877) 457-3760**, or via email, at visioffice@essilorusa.com.

